


Aspen LSO

Doctor: _____ **Fitter:** _____
Patient Name: _____ **Date:** _____
Patient #: _____ **Additional Follow-Up Dates:** _____

TOOLS NECESSARY: Tape Measure

THREE STEPS TO SIZING LSO/TLSO

- 1. Determine LSO/TLSO Size (Short or Tall)**
Measure the distance from the sternal notch to navel.
- 2. Determine Side Panel Circumference**
Average the lower rib and hip circumferences.
- 3. Determine TLSO Upgrade Kit Size**
Use the measurement from Step 1.

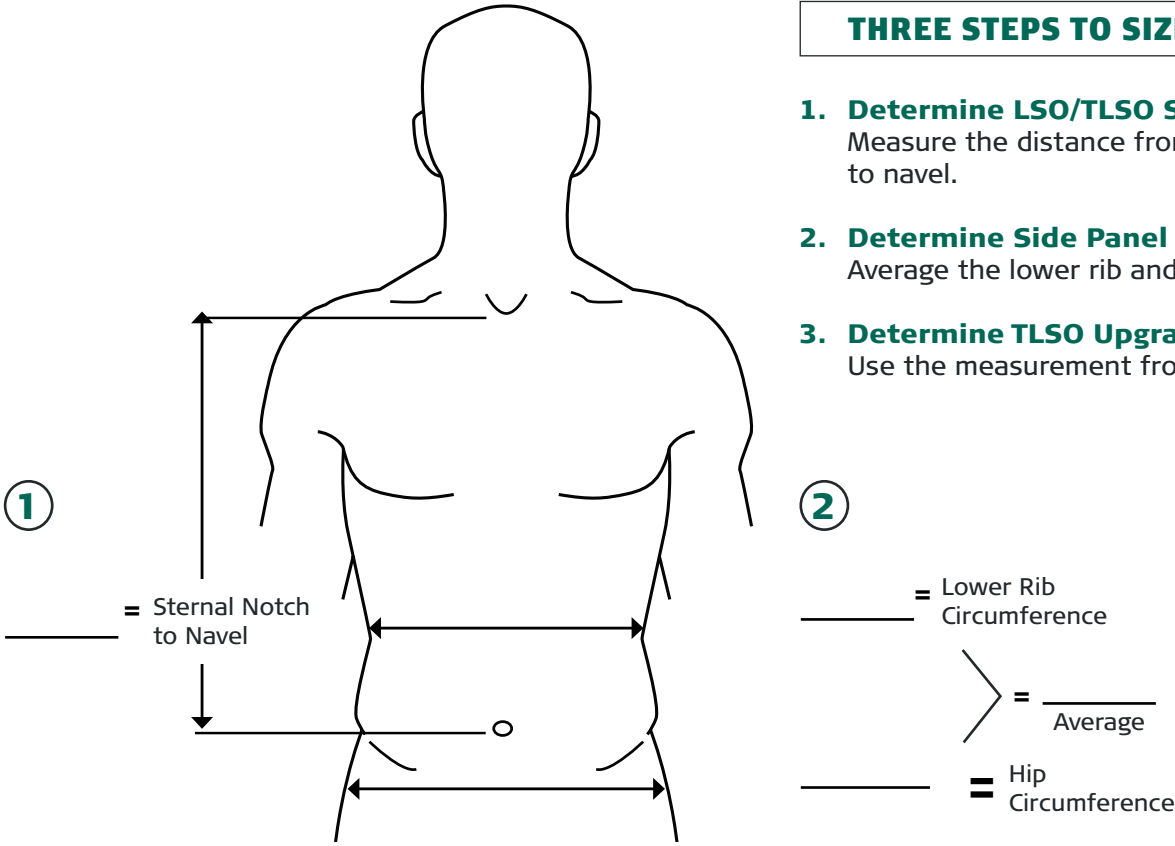

STEP 1 - BACK PANEL	STEP 2 - SIDE PANELS	STEP 3 - UPGRADE KIT
<input type="checkbox"/> <14.5 in (<37 cm) = Short 13 in (33 cm)	<input type="checkbox"/> 26-39 in (66-99 cm) = Small	<input type="checkbox"/> 11-13.5 in (28-34 cm) = X-Short
<input type="checkbox"/> ≥14.5 in (≥37 cm) = Tall 16 in (41 cm)	<input type="checkbox"/> 36-49 in (91-124 cm) = Large	<input type="checkbox"/> 12.5-16 in (32-40 cm) = Short
	<input type="checkbox"/> 32-47 in (81-119 cm) = Small	<input type="checkbox"/> 14.5-18 in (37-46 cm) = Tall
	<input type="checkbox"/> 44-59 in (112-150 cm) = Large	

Aspen TLSO	CAT NO	Aspen LSO	CAT NO	TLSO UPGRADE KIT	CAT NO
<input type="checkbox"/> Short / Small	991030	<input type="checkbox"/> Short / Small	990040	<input type="checkbox"/> X-Short	991053
<input type="checkbox"/> Short / Large	991031	<input type="checkbox"/> Short / Large	990041	<input type="checkbox"/> Short	991051
<input type="checkbox"/> Tall / Small	991070	<input type="checkbox"/> Tall / Small	990060	<input type="checkbox"/> Tall	991050
<input type="checkbox"/> Tall / Large	991071	<input type="checkbox"/> Tall / Large	990061		