ASPEN CONTOUR™ TLSO SPK

THORACIC LUMBAR SACRAL ORTHOSIS
LUMBAL-SAKRAL-THORAKALE ORTHESE
ORTESIS TORÁCICA LUMBOSACRA

ORTHÈSE THORACIQUE LOMBO-SACRÉE
LUMBALE SACRALE THORACALE ORTHÈSE
BEDERNÍ KRÍŽOVÁ HRUDNÍ ORTÉZA
ΛΕΝΔΗΦΥΡΒΛΕΤΗΛΗΝ ΘΟΡΑΧΟΡΤΗΣΗ

Center the RAP and chest panel. Slide the RAP on the adjustment strap on the inside of the QuikDraw as needed to fit the midline.

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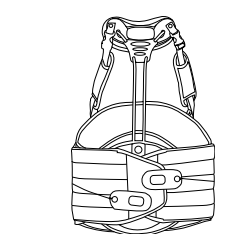
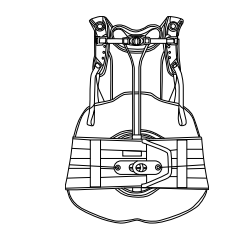
ÓRTESE TORÁCICA SACRO LOMBAR

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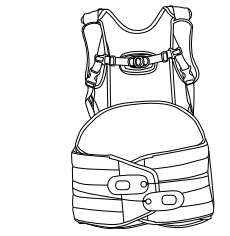
ASPEN CONTOUR TLSO SPK COMPLETE

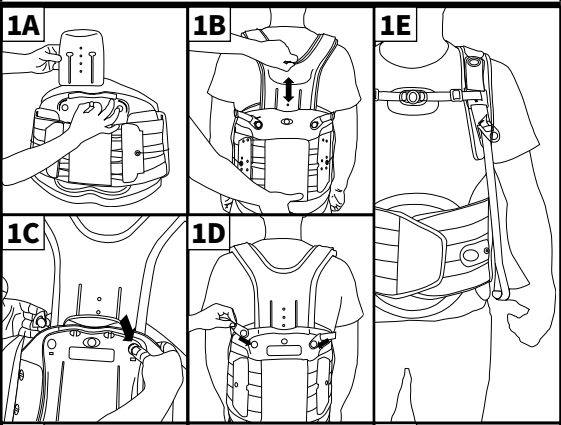
ASPEN CONTOUR TLSO UPGRADE CHEST PAD

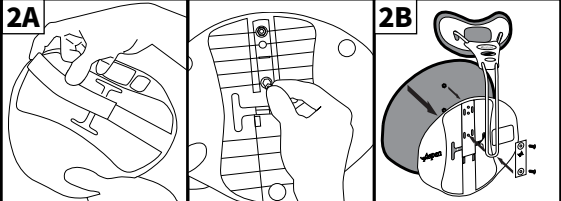
ASPEN CONTOUR TLSO UPGRADE SHOULDER STRAP

MD MEDICAL DEVICE
SINGLE PATIENT - MULTIPLE USE
CAUTION
EMERGO EUROPE
Westervoortsteed 60, 6827 AT Arnhem, The Netherlands

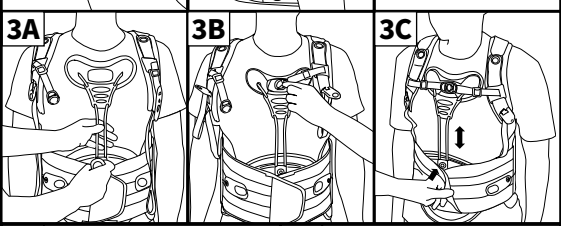
Pat. aspenmp.com/patents
Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200

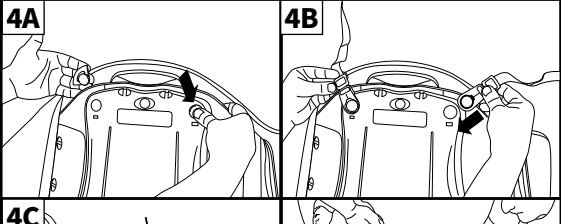
ENGLISH
1. TLSO STRAP INSTALLATION
For Contour LSO instructions, please log onto www.aspenmp.com.
A. Turn the yellow rivet to release the extension panel. Pull the panel out.

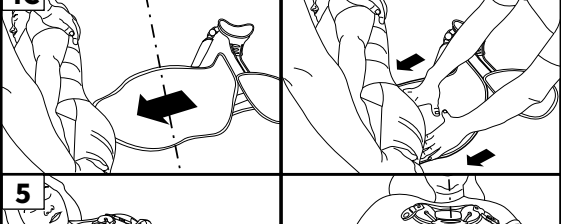
2. RAP & SPK PREPARATION
A. Remove pad from the back of the RAP and remove the cap from the front of the RAP. Flip the RAP over and press the nuts into the openings.
B. Attach the chest panel into the groove in the RAP. Insert and tighten the two screws to hold it in place, but loose enough to allow for the SPK.

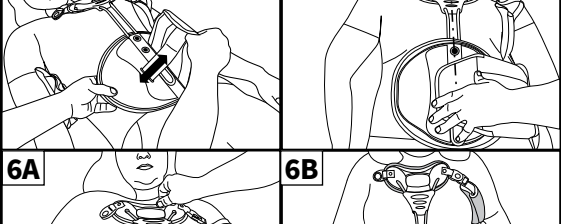
3. CHEST PANEL ADJUSTMENT
A. Find the correct height and tighten the screws to lock the chest panel in place.
B. Slide the shoulder straps into the Contour panel. Adjust to the proper height setting and then lock in place with the yellow rivet.

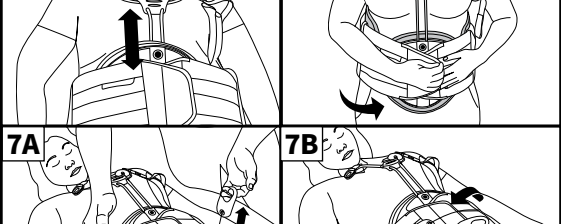
4. CONTOUR TLSO SPK PREPARATION
The Contour TLSO with Sternal Pad Kit Option is used to support and stabilize the thoracic lumbar-sacral portion of the spine for both non-operative and post-operative indications.

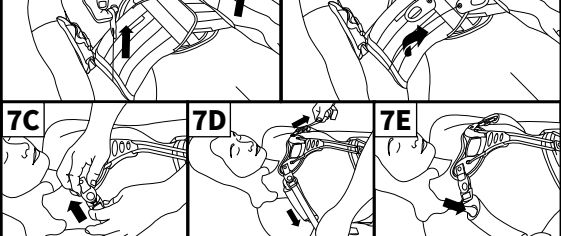
5. SPK PLACEMENT
Center the RAP and chest panel. Slide the RAP on the adjustment strap on the inside of the QuikDraw as needed to fit the midline.

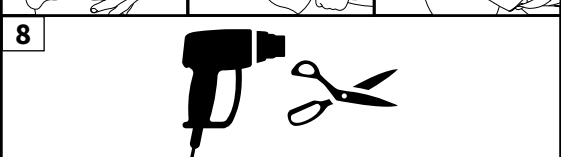
6. SPK ADJUSTMENT
A. Slide the chest panel to the desired height and tighten the screws to secure in place. Flip the RAP over and press the nuts into the openings.
B. Attach the chest panel into the groove in the RAP. Insert and tighten the two screws to hold it in place, but loose enough to allow for the SPK.

7. SPK TIGHTENING
A. Peel the straps away from the brace, and pull out and away to tighten.
B. When tightened, place the pull-tabs down onto the belt and press down to secure.
C. Clip the SPK strap into the chest panel.
D. Pull down on the SPK straps to tighten.
E. Extra strap can be tucked into the pad on the shoulder strap.

8. MODIFICATION OPTIONS
This product may be substantially modified by an individual with expertise. For more information, please visit www.aspenmp.com/education/product-modifications.

8. MODIFICATION OPTIONS
This product may be substantially modified by an individual with expertise. For more information, please visit www.aspenmp.com/education/product-modifications.

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INDICATIONS
Postoperative support, Degenerative disc disease, Bulging or herniated disc, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Kyphosis, Spondylolysis (Osteoarthritis/Degenerative Joint Disease).

CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders.

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