

## STRAP ACCESSORY KIT & EXTENDED STRAP

ZUBEHÖRSATZ FÜR GURT UND GURTVERLÄNGERUNG  
KIT DE ACCESORIOS DE CORREA Y CORREA EXTENDIDA  
KIT DI CINGHIE ACCESSORIE E CINGHIA ESTESA  
KIT D'ACCESSOIRES DE SANGLE ET SANGLE LONGUE  
RIEM ACCESSOIRE KIT & VERLENGDE RIEM  
SADA PŘÍSLUŠENSTVÍ K POPRUHU A PRODLOUŽENÝ POPRUH  
STROPPE TILBEHÖRSÆT & UDVIDET STROPPE  
ΕΞΟΠΛΙΣΜΟΣ ΕΞΑΡΤΗΜΑΤΩΝ ΛΟΥΡΙΟΥ & ΕΚΤΕΝΣ ΛΟΥΡΙ  
STROPPE TILBEHÖRSSETT OG UTVIDET STROPP  
ZESTAW AKCESORIÓW DO PASÓW I DO ICH PRZEDŁUŻENIA  
HIHNALISÄVARUSTEET & JATKOHIHNA  
REMTILLBEHÖRSSATS OCH FÖRLÄNGD REM  
НАБОР ПРИНАДЛЕЖНОСТЕЙ И УДЛИНИТЕЛЬНЫЙ РЕМЕНЬ  
KIT DE ACESSÓRIOS DE CORREIA E CORREIA ESTENDIDA  
扣帶配件包和扩展扣帶

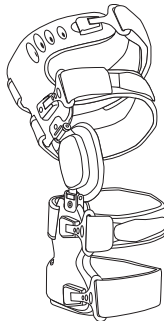ASPEN. OA KNEE+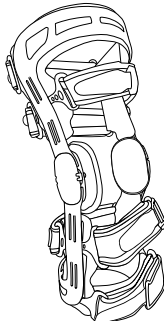

ASPEN. CI KNEE

 MEDICAL DEVICE

 SINGLE PATIENT - MULTIPLE USE


 CAUTION

   **EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands


 Aspen Medical Products, LLC  
6481 Oak Canyon, Irvine, CA 92618-5202 USA  
aspemp.com • +1-800-295-2776 • +1-949-681-0200  
©2021 Aspen Medical Products, LLC. All Rights Reserved.

LB0048C revised 06/2021

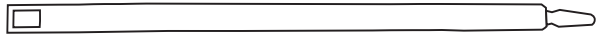
ASPEN OA KNEE +

EXTENDED STRAP  
GURTVERLÄNGERUNG  
CORREA EXTENDIDA  
CINGHIA ESTESA  
SANGLE LONGUE  
VERLENGDE RIEM  
PRODLOUŽENÝ ŘEMEN  
UDVIDET STROPPE  
ΕΚΤΕΝΣ ΛΟΥΡΙ  
UTVIDET STROPP  
PRZEDŁUŻENIE PASÓW  
JATKOHIHNA  
FÖRLÄNGD REM  
УДЛИНИТЕЛЬНЫЙ РЕМЕНЬ  
CORREIA ESTENDIDA  
扩展扣帶

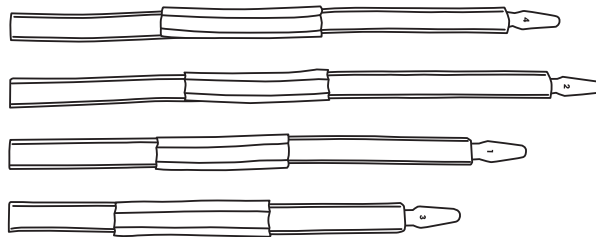ASPEN OA KNEE +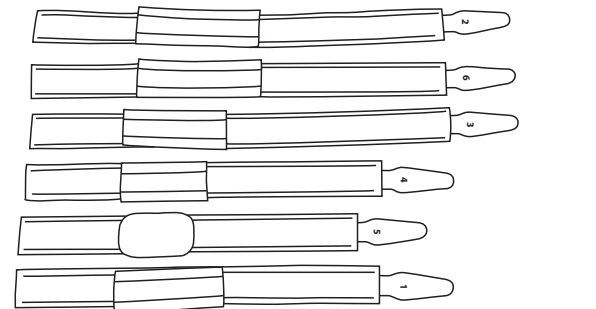

ASPEN CI KNEE

STRAP ACCESSORY KIT  
ZUBEHÖRSATZ FÜR GURT  
KIT DE ACCESORIOS DE CORREA  
KIT DI ACCESSORI PER CINGHIE  
KIT D'ACCESSOIRES DE SANGLE  
RIEM ACCESSOIRE KIT  
SADA PŘÍSLUŠENSTVÍ POPRUHU  
STROPPE TILBEHÖRSÆT  
ΕΞΟΠΛΙΣΜΟΣ ΕΞΑΡΤΗΜΑΤΩΝ ΛΟΥΡΙΟΥ  
STROPPE TILBEHÖRSSETT  
ZESTAW AKCESORIÓW DO PASÓW  
HIHNALISÄVARUSTEET  
REMTILLBEHÖRSSATS  
НАБОР АКССЕСУАРОВ ДЛЯ РЕМНЯ  
KIT DE ACESSÓRIOS DE CORREIA  
扣帶配件包

ENGLISH

### INTENDED USE

Not applicable.

### INDICATIONS

Not applicable.

### CONTRAINDICATIONS

Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

### PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These direccuads are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

### CARE INSTRUCTIONS

Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

### MATERIALS

Nylon, PVC, Polyurethane.

### NOT MADE WITH NATURAL RUBBER LATEX.

### DISPOSAL

Product may be safely disposed in accordance with local laws.

### REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

### LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:

- The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
- The product is altered;
- Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
- The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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Aspen Medical Products, LLC heeft geen enkele verplichting onder deze beperkte garantie in het geval:
- Het product niet gekocht is bij Aspen Medical Products, LLC of via de geautoriseerde distributiekanalen;
- Onderdelen in het product worden geplaatst die niet door Aspen Medical Products, LLC worden geleverd; of
- Het product niet wordt gebruikt in overeenstemming met de gebruiksvoorschriften van Aspen Medical Products, LLC.