
ASPEN SIERRA UNIVERSAL COLLAR™

- CERVICAL COLLAR
- ZERVIKALORHESSE
- CUELLO CERVICAL
- COLLARE CERVALE
- COLLIER CERVICAL
- NEKBRACE
- KRČNIŃ LÍMEC
- HALSHVIRVELKRAVE
- АУХЕНИКО КОЛАРО
- CERVICAL KRAGE
- SZYJNY KOLNIERZ ORTOPEDYCZNY
- КАУЛАТУКИ
- НАСКРАЖЕ
- ШЕЙНЫЙ КОРСЕТ
- COLAR CERVICAL
- 頸托





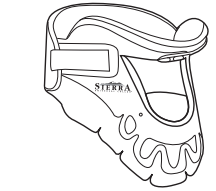
ASPEN SIERRA ADULT UNIVERSAL







ASPEN SIERRA CHILD

ADULT UNIVERSAL

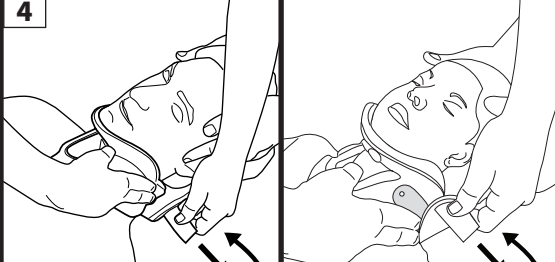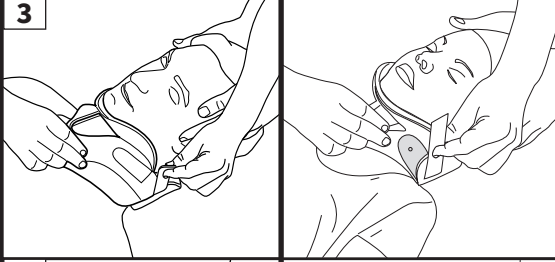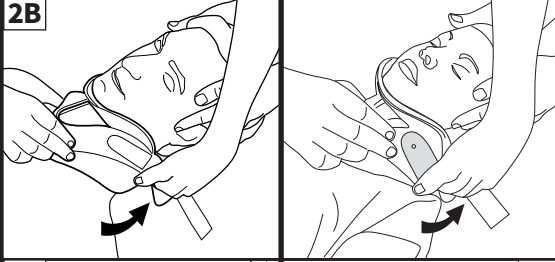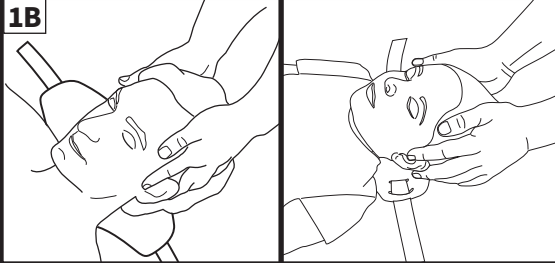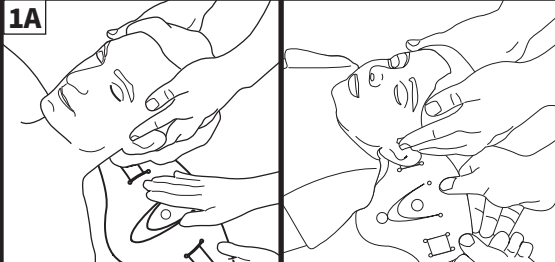CHILD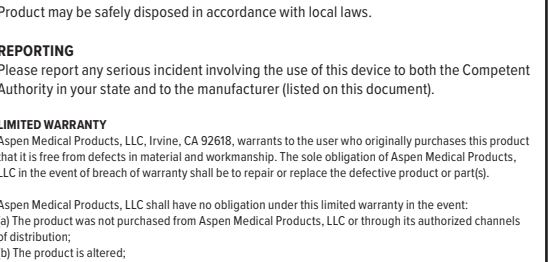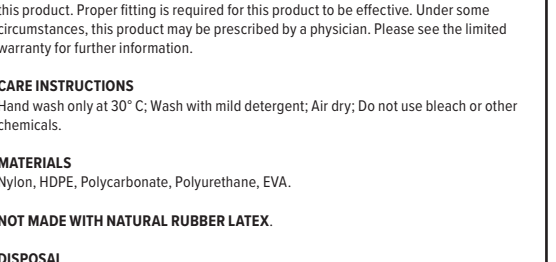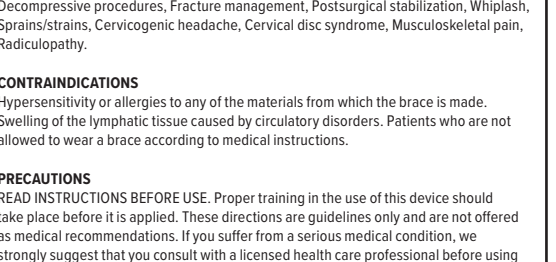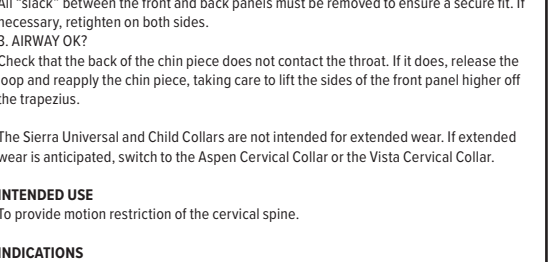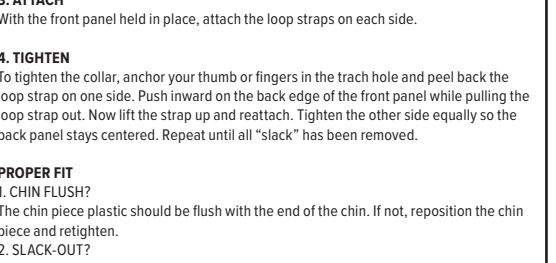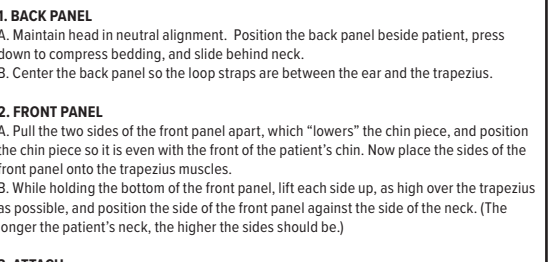

1. BACK PANEL
A. Maintain head in neutral alignment. Position the back panel beside patient, press down to compress bedding, and side behind neck.
B. Center the back panel so the loop straps are between the ear and the trapezius.

2. FRONT PANEL
A. Pull the two sides of the front panel apart, which "lowers" the chin piece, and position the chin piece so it is even with the front of the patient's chin. Now place the sides of the front panel onto the trapezius muscles.
B. While holding the bottom of the front panel, lift each side up, as high over the trapezius as possible, and position the side of the front panel against the side of the neck. (The longer the patient's neck, the higher the sides should be.)

3. ATTACH
With the front panel held in place, attach the loop straps on each side.

4. TIGHTEN
To tighten the collar, anchor your thumb or fingers in the trach hole and peel back the loop strap on one side. Push inward on the back edge of the front panel while pulling the loop strap out. Now lift the strap up and reattach. Tighten the other side equally so the back panel stays centered. Repeat until all "slack" has been removed.

2A
All "slack" between the front and back panels must be removed to ensure a secure fit. If necessary, retighten on both sides.

3. AIRWAY OK?
Check that the back of the chin, piece does not contact the throat. If it does, release the loop and reapply the chin piece, taking care to lift the sides of the front panel higher off the trapezius.

The Sierra Universal and Child Collars are not intended for extended wear. If extended wear is anticipated, switch to the Aspen Cervical Collar or the Vista Cervical Collar.

INTENDED USE
To provide motion restriction of the cervical spine.

INDICATIONS
Decompressive procedures, Fracture management, Postsurgical stabilization, Whiplash, Sprains/strains, Cervicogenic headache, Cervical disc syndrome, Musculoskeletal pain, Radiculopathy.

CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
Hand wash only at 30° C. Wash with mild detergent; Air dry. Do not use bleach or other chemicals.

MATERIALS
Nylon, HDPE, Polycarbonate, Polyurethane, EVA.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
Product may be safely disposed in accordance with local laws.

REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OR, OR DEFECT IN THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND DISCLAIMED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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