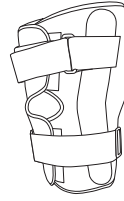
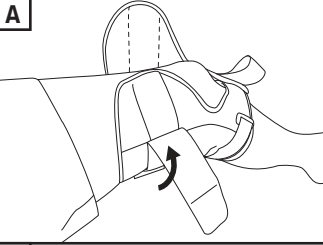
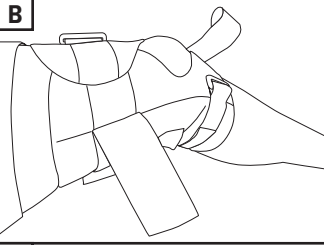
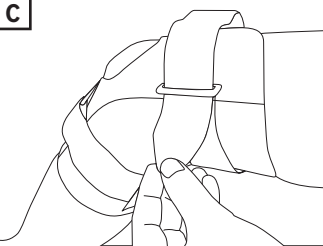
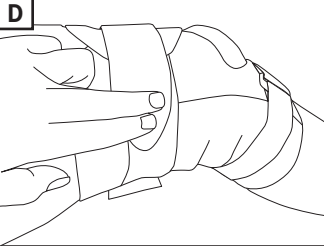
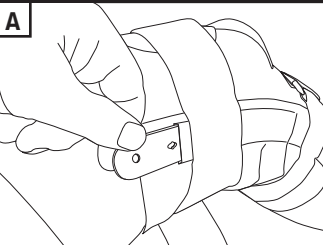
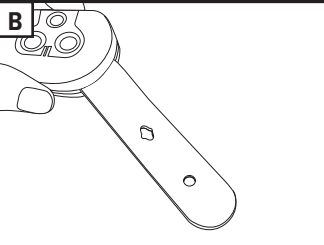
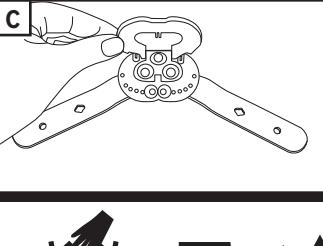
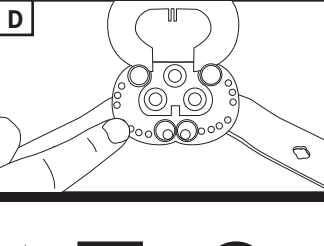
ASPEN SIZED ROM KNEE

ASPEN SIZED ROM KNEE

ASPEN MEDICAL PRODUCTS, LLC
 6481 OAK CANYON, IRVINE, CA 92618-5202, USA
 800-295-2776 · 949-681-0200 · aspenmp.com

		ENGLISH	ESPAÑOL	DEUTSCH	ITALIANO	FRANÇAIS	中文
<div style="display: flex; justify-content: space-around;"> <div style="border: 1px solid black; padding: 5px;"> A </div> <div style="border: 1px solid black; padding: 5px;"> B </div> </div>		<p>SIZING A. Begin 6 inches from the middle of the patella. B. Measure the circumference of the leg to identify the appropriate size.</p>	<p>TALLAS A. Comience a 6 pulgadas (15 cm) del centro de la rótula. B. Mida la circunferencia de la pierna para identificar el tamaño adecuado.</p>	<p>GRÖßEN A. Beginnen Sie 6 Zoll (15 cm) von der Mitte der Patella aus. B. Messen Sie den Umfang des Beins, um die geeignete Größe zu ermitteln.</p>	<p>DIMENSIONAMENTO A. Iniziare a circa 6 pollici (15 cm) dal centro della rotula. B. Misurare la circonferenza della gamba per identificare la taglia appropriata.</p>	<p>TAILLES A. Commencez à 6 pouces (15 cm) du milieu de la rotule. B. Mesurez la circonférence de la jambe pour identifier la taille appropriée.</p>	<p>尺寸调节 A. 从距离膝盖骨中心 6 英寸 (15 厘米) 的位置开始。 B. 测量腿围, 以确定合适的尺寸。</p>
	SIZING	X-SMALL	SMALL	MEDIUM	LARGE	X-LARGE	XX-LARGE
	11 - 15 in 28 – 38 cm	15 - 18 in 38 – 46 cm	18 - 21 in 46 – 53 cm	21 - 24 in 53 – 61 cm	24 - 27 in 61 – 69 cm	27 - 35 in 69 – 89 cm	
	Not made with natural rubber latex.	No está hecho con látex de caucho natural.	Ohne Naturkautschuklatex.	Non realizzato con lattice di gomma naturale.	Non fabriqué avec du latex de caoutchouc naturel.	并非采用天然胶乳制造。	

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LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS, LLC FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS, LLC SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.